



## PROCEDURE

### Materials Supplied

Calibrators should be used as specified in the **K-ASSAY**® Lp(a) immunoturbidimetric assay package insert.

Lyophilized Human Serum                      1 x 1 mL  
Diluent                                                      2 x 2 mL

### Materials Required But Not Supplied

**K-ASSAY**® Lp(a) immunoturbidimetric assay

Purified water

### Details of Procedure

Note: Allow all reagents and specimens to come to room temperature. Mix all reagents gently before using.

The **K-ASSAY**® Lp(a) Calibrators are assayed using the same procedure as the samples run in the test procedure. See package insert from the **K-ASSAY**® Lp(a) immunoturbidimetric assay.

### CALIBRATOR VALUES (Lot E744)

After reconstitution of the vial of lyophilized human serum with 1 mL of purified water, the Lp(a) concentration indicated on the vial label will be obtained.










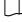

Lp(a) (mg/dL)
174.0

After reconstitution and dilution:

Calibrators A-F, values given in mg/dL						
	A	B	C	D	E	F
Lp(a)	0.0	17.4	34.8	69.6	104.4	174.0

The expected value for the **K-ASSAY**® Lp(a) Calibrator is continually being revised through ongoing quality assurance. As a result, the expected values may change from lot to lot. Please refer to the package insert for each lot for the exact calibrator value.

## LABELING SYMBOLS

	Lot Number
	Reagent
	Calibrator
	Expiration or "Use By" Date
	Catalog Number
	For <i>In Vitro</i> Diagnostic Use
	2-8°C Temperature Limitation. Store between 2 and 8 degrees C
	Potential Human Biohazard
	Manufacturer
	Consult Package Insert for Instructions for Use
	Authorized Representative in the European Community

## EU AUTHORIZED REPRESENTATIVE





### Advena Ltd.

Tower Business Centre, 2<sup>nd</sup> Flr.,  
Tower Street, Swatar, BKR 4013 Malta

## ORDERING / PRICING / TECHNICAL INFORMATION



### KAMIYA BIOMEDICAL COMPANY

12779 Gateway Drive  
Seattle, WA 98168 USA  
TEL: (206) 575-8068 / (800) 526-4925  
FAX: (206) 575-8094

Printed June 2017